



2020 universal premium professional device "LUX" © for permanent make - up with hygiene modules

The only device on the European market with which you can work without surface anesthesia!

It is a device for non-invasive treatment using skin-friendly technology of the latest technology, which is used in both the medical and cosmetic fields and is one of the most innovative and powerful devices on the European market. only the stretches can be obtained from the manufacturer and partner!

USPs

- ◆ Manufacturer "Die - Profi - Permanent - Liner" according to the Product Liability Act (BGBl. I p. 2198). We meet all the requirements required in the EU.
- ◆ It is unique and branded device. Pay attention to the device number and the company label on the device.
- ◆ Innovation according to the latest technical standards controlled by micro processors. Operate either on the mains or wirelessly via a lithium-ion battery of the latest generation.
- ◆ It convinces with precision and smoothness. No more vibration of the needle or modules, which means clean work. With the control unit you can work in 4 different power levels, with the lithium-ion accumulator (approx. 3 hours running time) in 3 different power levels directly on the device.
- ◆ The modules are designed in such a way that no color or active ingredients to be introduced into or onto the device, they are EO sterilized and can therefore be used even in sterile areas of a clinic.
- ◆ The modules are immediately fixed by a bayonet lock by turning them briefly.

modules

1/3/5 round or 5/7 flat needles suitable for all work without anesthesia!

The modules are EO sterilized and can be used in sterile clinic zones (disposable products).

They have a special needle guide so that safe and clean work (gamma sterilized, disposable items). Prevents the return of infected paint with bacteria and thus the infection and blockage of the handpiece u. no lymph gets in or on the device.

Automatically retrieves the needle when the device is switched off!

Use only original, otherwise the device can be damaged and the guarantee is void.

Applications and work

- ◆ Fine hair drawing, ideal lip shaping, correcting faulty permanent make - up and medical pigmentation.
- ◆ The module is filled with paint, there is no annoying immersion and you also save paint.
- ◆ You can work with several colors at the same time and only need to change the module, so it is no longer necessary to work with two handpieces.

Technical specifications:

- ◆ weight 60 grams.
- ◆ Motor: Swiss Motor noise level less than 30 dB
- ◆ Handpiece made of highly refined metal / aluminum, color gold.
- ◆ Rechargeable lithium-ion battery Voltage: 5 V / 1 A / long operating time when working.
- ◆ Control unit / needle stroke 4 stages 15000-35000 u. Lithium-ion battery 3 levels 35000 / minute adjustable.
- ◆ By hand on the lithium-ion battery or handpiece. off switchable!

Scope of delivery:

- ◆ 1 universal premium device "LUX" © for permanent make-up
- ◆ 1 external control unit suitable for USB or mains connection.
- ◆ 1 CE power supply for charging the lithium-ion battery and the control unit.
- ◆ 2 adapter cables for the lithium-ion battery and control unit
- ◆ 1 box of 10 modules of 5 and 7 flat

These pages are the sole property of "Die-Profi-Permanent-Liner", copies or duplications are not permitted without permission.
 In the event of misuse, a notification with an application for damages is made immediately!

device Description

Pressure switch for switching on / off and changing the 3 switching levels

Connection socket adapter cable for the lithium-ion battery

and charging via the power supply / charging adapter CE

Lithium-ion battery

Company name and device no.

Scale for adjusting the needle depth

Hygiene module PMU bayonet lock

Power connection socket via the power supply

External control unit suitable for work via mains connection

Needle lift control stage 1 - 4

Setting ring for levels 1 - 4 needle stroke

On-off switch

Pressure switch for switching on / off and changing the 3 switching levels

Power supply / charging adapter CE for mains operation with control unit



These pages are the sole property of "Die-Profi-Permanent-Liner", copies or duplications are not permitted without permission.
In the event of misuse, a notification with an application for damages is made immediately!

Installation

1. When working with the control unit on the mains, unscrew the lithium-ion accumulator and screw the control unit onto the device. Plug the EU power supply unit supplied with the device into the socket or USB port. the jack plug into the device. Caution, do not use any other power supply, otherwise the guarantee may be void.
2. To switch off, press the operating mode switch gently for approx. 10 seconds. The electronic control is then started or switched off!
3. There is an adjustable scale on the lower part of the device. You can adjust the needle depth here by turning it slightly to the right or left.
4. The module is immediately fixed by a short turn to the right (bayonet lock) and can be removed by turning it slightly to the left. can be exchanged, e.g. with permanent make - up, work with several colors without 2 handpieces at a time! The original modules must be used to ensure that the device functions properly. Otherwise the guarantee can be void!
5. When the device is on display, the needle does not protrude, so that there is no risk of unintentional injury when the module is attached or removed.
6. Working with the lithium-ion battery. First unscrew the control section, then unscrew the charged lithium-ion battery. When the lithium-ion battery is empty (running time approx. 3 hours), unscrew it from the device and charge it via the adapter cable either at the mains or USB connection.
7. The device may only be used by trained specialist personnel!
8. Ensure that the device never falls down, it is appropriate precision engineering that is not suitable as a projectile!
9. When disinfecting the device with disinfectants, the instructions for use of the disinfectant manufacturer must be observed, they must not detach paint, metals and plastic. Otherwise, the warranty may otherwise expire!

Warranty conditions Factory

warranty: The statutory warranty according to § 437 BGB is 24 months as a commodity, it means that the devices and products delivered by us are free from defects in processing and material and begins with the delivery of the goods (see accounting). It refers to all devices and products supplied by us, excluding consumables. During the warranty period, if the device fails, you can send in the defective device and confirm this by filling out the form in the shop <https://www.die-profi-permanent-liner.de/defektes-geraet>, and you will receive promptly in the event of a warranty / guarantee Exchange for a new product / device. Exception here, the device is damaged by improper use! The replacement is not free of charge here. You will then be given a price offer for exchange, which is up to a maximum of € 500 net. A prerequisite for the warranty is that the device was purchased from us, see device number.

Are excluded from the guarantee

Batteries and lithium-ion accumulators as well as damage due to improper use / non-compliance with the safety instructions / unauthorized interference. Please make sure to pack the device appropriately. We are not liable for damage that occurs during transport. After the warranty period has expired, you also have the option of sending the defective device to us for repair (the defective product must also be registered via the shop <https://www.die-profi-permanent-liner.de/defektes-geraet>). The manufacturer assumes responsibility for the safety, reliability and performance of the product only in the event that the repair, replacement or modification of the device is carried out by authorized persons. Furthermore, he assumes responsibility if the device is used properly in accordance with the instructions for use and the performance specifications (electricity) meet the legal requirements. Subject to changes in technical data, properties, typesetting and printing errors.

We give an additional guarantee to the "LUX" © series delivered by us. H. Your guarantee is valid for a total of 3 years! As a customer The buyer of the device, the replacement takes place after the warranty period, or in the event of wear and tear against a new device for a fee of € 250 net plus 19% VAT.