

CoolMax 360



Manufacturer according to ProdHaftG (BGBI. I p. 2198) / Made in Germany
Training and Distribution Equipment Accessories for Non-Invasive Aesthetic Treatments * Shock Therapy * Cryolipolysis Medical
& Cosmetic Permanent Make-up * Photo Therapy * Nano - Needling * Plasma Lifting
Bahnhofstraße 32 * 12207 Berlin * Phone 030/ 37308867 * Fax: 030 89408506 * Radio telephone 015254276587
Mail: die-profi-permanent-liner@onlinehome.de * Online shop: <http://www.die-profi-permanent-liner.de/>

Kryolipolyse Gerät Cool Max 360 VK: 5.300,00 € inkl. 19 % MwSt.

Please read through the manual before operating the system

this product does not belong to medical ,please don't use for disease treatment !



CoolMax 360

CoolMax 360

1. Safety Notice

- 1.1 Please don't disassemble the equipment or try to do some other operation, which haven't been indicated in our instruction. All the repairing work should be done by our admitted professional personnel
- 1.2 Please don't assemble or operate this equipment while you are near the water or your hands are wet. Notice that don't pour the liquid on it.
- 1.3 Please pull out the plug from the socket when you met such conditions as below. And in touch with professional maintenance Station. A. The equipment touched liquid B. The equipment has abnormal smell, smoking or loud and strong noise. C. Cable is broken D. The equipment is fallen down and broken
- 1.4 Please don't put anything on the cables, don't place the equipment at any place where someone can step on the cable.
- 1.5 Please pull out the power cable after using it for your safety
- 1.6 Don't put anything into the equipment's jack, otherwise it may cause fire or touch the electricity. If something enters the equipment, please don't pick it out by yourself, contract distributor or our company immediately
- 1.7 Please don't put the equipment on the anything that can't stand steadily, avoid it turns over and damage.
- 1.8 Our company has the right to modify the information in this introduction before notify! And keep the explanations right for the above.

2. Operation Notice

Before operation, please read the following content.

- 2.1 It is essential to make different setting according to specific treatment demand.
- 2.2 The application of The machine must never be painful and cause redness to the skin or leave marks.
- 2.3 Do not apply the device on children less than 7 years old. The treatment to children over 7 requires medical advice.
- 2.4 Do not use this device if suffering from a fever, inflammation, dermatitis (acne, eczema, herpes), warts, open or recently stitched wounds, serious and progressive illnesses (cancer, AIDS, hepatitis, blood diseases, etc. or blood coagulation problems. Do not apply the device if you have or have had phlebitis, after veins surgery within the last two months, if you have varicose veins or tingling in your limbs, if you have any concerns consult a physician.
- 2.5 Never use the device on a pregnant woman.
- 2.6 The machine could be used on all external and healthy areas of the body except the eyes, throat, ears, glottis and the genital area.
- 2.7 Must not be applied inside the mouth, or on mucous membrane, or on nipples, or moles without medical advice.
- 2.8 Never use the device if you suffer from epilepsy. The device must not be used by any person suffering from mental illness.
- 2.9 In the case of long-term treatment with drugs such as anti-inflammation, anticoagulants,

CoolMax 360

antibiotics or if your skin bruises at the slightest shock, ask advice of your doctor/physician before using.

2.10 Never apply to the body parts with jewelry or ring.

2.11 Regular use of The machine enhances well-being; it is better not to stop its use for a prolonged period of time. Not following operating instructions and usage recommendations may bring lesser results than expected. Do not use the device longer than the recommended time.

2.13 If your symptoms have no improvement, or if in any doubt on the use, consult your doctor.

2.12 Effect may vary depending on your age, life habits (level of physical activity, diet, etc.).

2.13 The machine can be used after any plastic surgery only while following the instruction of the surgeon.

2.14 The machine is designed for professional use only, any personal use of this device is prohibited.

2.15 The Machine continuous working time shall not exceed 1 hours. The best way is cut the power off, let the machine have a rest for 20-30mins. After the machine have worked for a hour.

2.18 Never use the strong acid or strong base or alcohol clean the machine. (use physiological saline to clean machine.)

How Cryolipolysis works



CoolMax 360

Cautions:

- 1.The machine must be operated by professionals.
- 2.Observe the status of Distilled water level window before operating each time, the level can't less than level 8.
- 3.Banned in no case of anti-freeze membrane treatment, during the course of treatment must ensure anti-freezing film completely cover the rapeutic part, otherwise can appear frostbite easily, etc.
- 4.According to the clinical, Individual physical users may occur frostbite reaction, you can raise the temperature according to fat thickness or consult professional person.

5.Cautions especially:

The person please do not treat a large area of body or treat the adjacent part of the body that have been treated in the same course of treatment. As (Fig17) shows: if you operate of part ①, you can not operate the part ② or part ④, so you can only operate part ③. If you against this regulation, Light will cause the metabolic disorders, Heavy then cause metabolic acidosis and the toxins from the body doesn't stranded.

- 6.Patients using the following medical electrical equipment can not use the system.

- Implanted cardiac pacemakers or other implanted electronic devices;
- Life-sustaining artificial heart-lung machines and other equipment;
- Portable ECG measurement equipment;

- 7.The following patients should be agreed by the professional doctors before using the device.


- Patients with acute disease;
- Malignancy;
- Patients with Infectious diseases;
- Pregnant women;
- Patients with heart diseases;
- Patients with fever;
- Patients are receiving treatment or those with abnormal body phenomenon;



Attention:

- 1.Please install Two Cryolipolysis hand pieces correctly before filling the water into the water tank.
- 2.Please operating with the help of professionals when operate the machine.

CoolMax 360

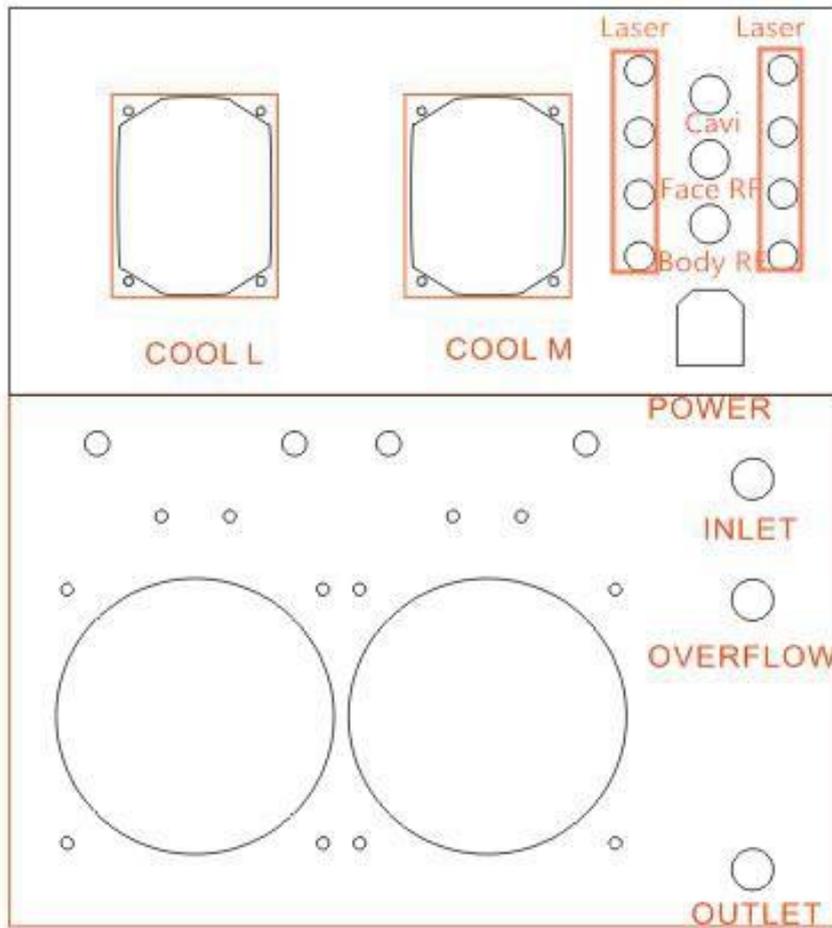

Applicator Description and Installation:

1. Emergency stop button and Key Switch:



Emergency Stop Button:

Press--EMG stop

Reset--Rotate 90° CW.

Key Switch:

Rotate 90° CW to turn the machine on.

NOTE: Ensure that the EMG-stop button is reset , and the key switch turn on before operating the machine.

CoolMax 360

1. Turn on the machine . Then you will see the display on like the following::



2. Large Cryolipolysis hand piece:with 4 size CUP

Combines the cryolipolysis, Vacuum and Photon Dynamic.

Take the cryolipolysis hand piece on the holder at the right side,then insert it into the H3,H4 socket.




3. Medium Cryolipolysis hand piece:with 3 size CUP

Combines the cryolipolysis, Vacuum and Photon Dynamic.

Take the cryolipolysis hand piece on the holder at the left side,then insert it into the H1,H2 socket.

CoolMax 360

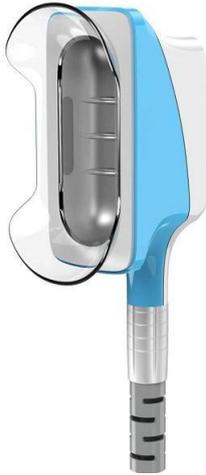


4.S size Cryolipolysis hand piece **for the double chine removal**

Combines the cryolipolysis, Vacuum.

Take the cryolipolysis hand piece on the holder at the left side, ONLY insert it into the H1 socket.



Cryolipolysis Hand pieces with 4.3 inch touch screen:

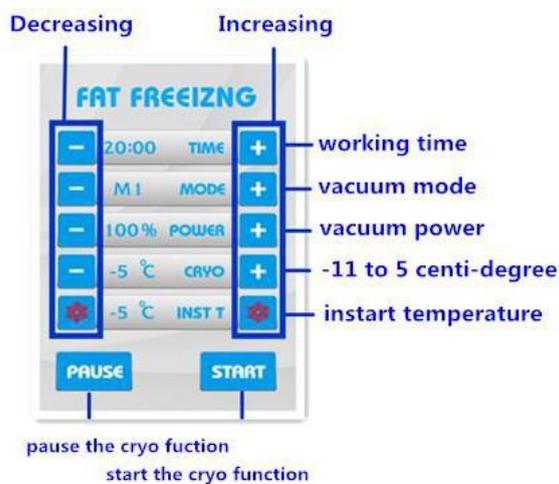

5.Suction Emergency Stop Switch:

CoolMax 360

It is designed to control the vacuum function,press it to release all the suction.

(please ask your patients to take it during the cryolipolysis treatment,if any emergency,then they could press it to release all the suction).

Insert it into the emergency bar socket.



6.cavitation Hand piece:

Take the cavitation hand piece on the relevant holder,then insert it into the cavitation 4 pins socket.



7.Radio frequency for Body :

Take the RF-body hand piece on the relevant holder,then insert it into the RF-body 6 pins socket.



8.Radio frequency for Face :



Take the RF-body hand piece on the relevant holder,then insert it into the RF-body 5 pins socket.

9.Lipo Laser hand pieces:

6 pads diode lasers.

Take the lipo laser pads on the holder top of the machine.then insert them into the laser socket.



10.the filter cleaning

When the liquid in the filter exceeds more than half of the bottle, please clean the filter immediately as following picture,

when you cleaning the filter,please don't miss any small parts.


CoolMax 360



Filling water into the water tank(recommended two times):

- 1.Insert two tubes into the water inlet hole and overflow hole.
 - 2.Filling the water into the water tank.
 - 3.When the overflow pipe flowing the water out,then stop filling the water.
 - 4.Star the cryolipolysis functions for about 2 mins.
 - 5.Then stop the machine and filling the water again like the step 1 till the overflow tube flow out the water then stop filling the water,till now filling water is finished.
- and you could do the cryolipolysis treatments now.

It is recommended that the water tank be filled again 1 time per 1 months.
(it also depends on the treatments times).

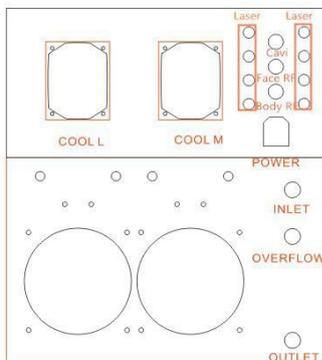

If you need to change the water,you need to drain the rest of water from
the water tank.

- 1.insert two tubes into the overflow and water outlet holes.then the water will be flowing out from the outlet tube.

Note:

CoolMax 360

1.If not using the device 15 days or above,we recommended to put the distilled water clean and clear

2.when there is not enough water in the water tank or no water flowing,then the adding

water reminder will display like following: then pls help to add enough water.



3. After-Sale Service

Warranty

Our factory stipulates the warranty for the original purchaser as follows:

- The term of maintaining guarantee is main device 1 year and spare part 3 months from the date of purchasing according to the receipt. (Expendable is not included in it.) Others not included in maintaining guarantee or over the time, we also supply to maintain and renew but need charges. Please contact with the dealer if need change of purchasing the parts and easily expendable products.
- According to the warranty stipulation, our responsibilities are limited to repair and replacement of the part and material recognized in conformity with above conditions.
- won't bear any responsibility for any damage caused by misapplication of the user or repair of disapproval

Out of Warranty

- A product that has no SERIES Number
- Applicators dropping or any shock
- Applicator acrylic rack

Maintenance Advantage

- Easy to check and confirm the problem
- Simple to find out the solution
- Quick to replace Function Module

Maintenance Policy

CoolMax 360

- Must provide the photo, even the Video depends upon the different situation

Without manufacturer's permission, purchaser can't send back the machine, otherwise, the purchaser has to bear the transportation fee and the customs tax and manufacturer has the right to refuse to handle.

Caution

Contraindication

Please avoid the treatment from the following patients

1. Pregnant women or women who may be pregnant
2. Cardiopaths
3. Patients with vascular diseases
4. Patients with pacemakers
5. Patients with thrombosis and or thrombophlebitis
6. Patients being treated with anticoagulants
7. Patients who have undergone a transplant
8. Carriers of large metal prostheses
9. Diabetics
10. Epileptics
11. Patients with cancer.
12. Patients with heart diseases.
13. Patients with sensory or physical impairment.

Caution when use

The machine should be

Cryolipolysis Function

- Recommended room temperature be 25 to 29 centidegree.
- Do use protection of GOOD QUALITY protection of frost film to prevent skin frostbite.

After one client treatment or 2 hours usage, please allow about 20 minutes resting of system, it can prolong the lifetime of machine.

CoolMax 360

4. Operation:

Main operation interface:



1. CRYO **CRYO** :touch it and enter into the Large Cryolipolysis& Vacuum&PDT function interface;

2. Cavitation **CAVI** :touch it and enter into the Cavitation function interface;

3. BODY RF **BODY RF** :touch it and enter into the lipo laser function interface;

4. FACE RF **FACE RF** :touch it and enter into the body rf function interface;

5. Lipo Laser **LASER** :touch it and enter into the Lipo Laser interface;

CoolMax 360

Operation:

Large & Medium Cryolipolysis



1.set the treatment time  :

Touch   to adjust the treatment time from 5 to 60 mins,(preset time:20 mins)

 :

Touch   to adjust the vacuum mode among M1,M2,M3,M4 modes.

3.set the vacuum power  :

Touch   to adjust the vacuum power level from 10% to 100% .

4.set the Cryo Level  :

Touch   to adjust the cryo level from -11 to 5 centidegree.

5.Touch  to heat 3 minutes,become red  ,Then continue cooling,become white 

Note:they system is designed 3 minutes heating massage program,but it

is cooling by default,if you need the heating program at the beginning of

thetreatment,pls touch  to change into  ; the temperature will be about 37-45 centi-degree,3 minutes later,the system will enter into

Cooling state automatically.

CoolMax 360

Operation:

Cavitation



1.set the treatment time

TIME

Touch **-** **+** to adjust the treatment time from 5 to 30 mins,(preset time:20 mins).

2.Set the cavitation mode

CAVI MODE

Touch **-** **+** to adjust the cavitation mode among Auto,M1,M2,M3 modes.

3.set the cavitation power

CAVI POWER

Touch **-** **+** to adjust the cavitation power level from 10% to 100% .

4.Start button

START

:touch it to start the treatment after all the parameter adjusted.

5.Pause button

PAUSE

:touch it to pause the treatment,if need to change working mode,power or treatment time during treatment,please touch it firstly,then change the parameters.

6.Back button

BACK

:touch it to return to the main function interface,it would be touched

after touching the pause button

PAUSE

7.Stop button

STOP

:touch it to stop the treatment.

CoolMax 360

Operation:

BODY-RF



1.set the treatment time **TIME** :

Touch **-** **+** to adjust the treatment time from 5 to 30 mins,(preset time:20 mins).

2.Set the BODY RF mode **RF MODE** :

Touch **-** **+** to adjust the cavitation mode among Auto,M1,M2,M3 modes.

3.set the BODY RF power **RF POWER** :

Touch **-** **+** to adjust the cavitation power level from 10% to 100% .

4.Start button

START :touch it to start the treatment after all the parameter adjusted.

5.When starting the RF treatment,pls keep stepping footswitch when doing the RF treatment

6.Pause button **PAUSE** :touch it to pause the treatment,if need to change working mode,power or treatment time during treatment,please touch it firstly,then change the parameters.

7.Back button **BACK** :touch it to return to the main function interface,it would be touched

after touching the pause button **PAUSE** .

8.Stop button **STOP** :touch it to stop the treatment.

CoolMax 360

Operation:

FACE-RF



1.set the treatment time **TIME** :

Touch **-** **+** to adjust the treatment time from 5 to 30 mins,(preset time:20 mins).

2.Set the BODY RF mode **RF MODE** :

Touch **-** **+** to adjust the cavitation mode among Auto,M1,M2,M3 modes.

3.set the BODY RF power **RF POWER** :

Touch **-** **+** to adjust the cavitation power level from 10% to 100% .

4.Start button

START :touch it to start the treatment after all the parameter adjusted.

5.When starting the RF treatment,pls keep stepping footswitch when doing the RF treatment

6.Pause button **PAUSE** :touch it to pause the treatment,if need to change working mode,power or treatment time during treatment,please touch it firstly,then change the parameters.

7.Back button **BACK** :touch it to return to the main function interface,it would be touched

after touching the pause button **PAUSE** .

8.Stop button **STOP** :touch it to stop the treatment.

CoolMax 360

Operation:

LipoLaser



1.set the treatment time  :

Touch   to adjust the treatment time from 5 to 30 mins,(preset time:20 mins).


2.Set the Laser mode : 

Touch   to adjust the cavitation mode among Auto,M1,M2,M3 modes.

3.set the Laser power  :

Touch   to adjust the cavitation power level from 10% to 100% .

4.Touch any plaser pad to select the it to work

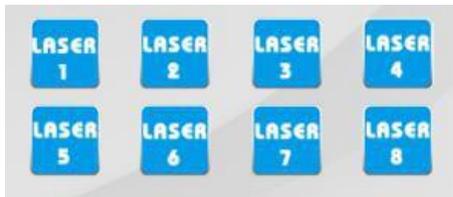

5.Start button



:touch it to start the treatment after all the parameter adjusted.

6.Pause button  :touch it to pause the treatment,if need to change working mode,power or treatment time during treatment,please touch it firstly,then change the parameters.

7.Back button  :touch it to return to the main function interface,it would be touched after touching the pause button .

8.Stop button  :touch it to stop the treatment.

CoolMax 360

Finish:

1. When finish the treatment, Press  button.
3. Turn off the power switch on the front of the machine and clean the accessories after treatment.

Contact Info

Repair or any enquiry for this, it shall be back to manufacturer

Finish:

1. Turn off the power switch on the front of the machine and clean the accessories after treatment.

Contact Info

Specification

Cryolipolysis temperature	5°C to -11°C adjustable
The material of Cryo handle	Medical Silicone
Heat temperature	37-45°C adjustable
Heat temperature time	3 minutes
Heat temp Advantages	avoid the frostbit during cryo treatment
Power	900W
Vacuum	650mmHg
Light& wavelength	Blue(480nm)
Machine Display	8.4 inch touch screen
Handle Display	4.3 inch touch screen
Cooling System	Semiconductor +water+air
L size Cryo handle	1 pieces with 4 size Cups
M size Cryo handle	1 pieces with 3 size Cups
Cryo Handle	5 handles cryo
L size Cryo handle	size 1:193*80*15mm size 2:210*90*35mm size3:215*100*40mm size 4:230*105*40mm
M size Cryo handle	size 1:140*73*15mm size 2:155*80*30mm size3:165*90*45mm
The cooling area of L size Cryo handle	175*70*45mm
The cooling area of M size Cryo handle	125*62*43mm
Cavitation frequency	40Khz
RF frequency	10Mhz
Lipo Laser Power	100mw/pcs
Lipo laser quantity	66pcs
Lipo Laser length	650nm
Power supply	110V-120V/220V-240V, 50/60Hz
Advantage	Two cryo handles can working at the same time

Warranty:**Warranty Conditions/ Warranty****Legal warranty according to § 437 BGB as consumer goods 24 months.****Factory Warranty:****As a customer of the company Die-Profi-Permanent-Liner you receive a factory warranty of 3 years when purchasing a device of the series "LUX" ©****What does a guarantee include?**

What does a guarantee include?

That the equipment and products supplied by us are free from defects in processing and material. It begins with the handover of the goods (see invoicing) and refers to all devices and products delivered by us, excluding consumables. During the warranty period, in the event of failure of the device after filling out the form in the shop: defective device and confirmation <https://www.die-profi-permanentliner.de/ticket/index/sFid/9> this submission and in the case of warranty / guarantee, you will promptly receive the exchange for a new product / device.

Exception, the device is damaged by improper use! Here the replacement is not free of charge.

You will be offered a price offer for exchange, which goes up to a maximum of 3,000, - € net.

Prerequisite for the warranty is, the device was purchased from us, see device number. Proof of the invoice. Excluded from the warranty are: Damage caused by improper use/ non-compliance with safety instructions/ by unauthorized intervention. Please make sure to pack the device appropriately. We are not liable for damages incurred during transport. Do Sir only work with the original consumables of the series "LUX" since these devices are equipped with corresponding controls via several microprocessors and the respective original consumables are hereby coordinated! If you do not © with the original consumables of the "LUX" series. work it can lead to damage to the device, i.e. failure of the control! Submitted devices are examined by the physicists working for us.

If the case turns out, the warranty / extended factory warranty expires for 3 years and also the goodwill as a customer and buyer of the device. Possibility to send the defective device for repair after registration via <https://www.die-profi-permanent-liner.de/ticket/index/sFid/9>).

We only assume responsibility for the safety, reliability and performance of the product as a manufacturer "Die - Profi - Permanent - Liner" in accordance with the Product Liability Act (BGBI. I p. 2198) if the repair, replacement or a Modification of the device is carried out by authorized persons (these are for devices of the series "LUX" © the specialist workshops working for us).

Technical data, characteristics, typesetting and printing errors are subject to change.

On the devices of the series "LUX" supplied by us © we give an additional warranty i.e. Your warranty lasts a total of 3 years!

As a customer and buyer of the device, the replacement also takes place after the warranty period, or in case of wear and tear against a new device for a fee of 3,000, - € net plus 19% VAT.